



**BIOLOX<sup>®</sup>** *delta* **TS**

**Revision Femoral Heads**



1. Ensure male taper of well-fixed femoral stem is wiped clean of any blood, bone chips or other foreign materials
  - **IT IS ESSENTIAL** that the taper on the stem be examined for damage
  - Refer to below examples for an acceptable level of damage to use ceramic BIOLOX® *delta* TS Revision Femoral Heads
2. Select the appropriate femoral head and place it onto the stem
3. Apply finger pressure to firmly seat the head onto the stem
4. Utilizing the femoral head impactor, impact the femoral head onto the stem with two moderate blows

**Acceptable Taper Damage**

Appropriate for use with BIOLOX *delta* TS Revision Femoral Heads

Undamaged Taper

Taper with scratches/warping

**Unacceptable Taper Damage**

Do not use with BIOLOX *delta* TS Revision Femoral Heads

Slanted Taper

Taper with broad truncation

Crushed Taper

### BIOLOX® *delta* TS Revision Femoral Head Articul/eze® 12/14

	Cat. No.
+1.5	1365-28-710
+5.0	1365-28-720
+8.5	1365-28-730
+12	1365-28-740

28 mm

	Cat. No.
+1	1365-32-710
+5	1365-32-720
+9	1365-32-730

32 mm

	Cat. No.
+1.5	1365-36-710
+5.0	1365-36-720
+8.5	1365-36-730
+12	1365-36-740

36 mm

	Cat. No.
+1.5	1365-40-710
+5.0	1365-40-720
+8.5	1365-40-730
+12	1365-40-740

40 mm

	Cat. No.
+1.5	1365-44-710
+5.0	1365-44-720
+8.5	1365-44-730
+12	1365-44-740

44 mm

**DePuy Orthopaedics, Inc.**  
 700 Orthopaedic Drive  
 Warsaw, IN 46581-0988  
 USA  
 Tel: +1 (800) 366 8143  
 Fax: +1 (574) 371 4865

**DePuy International Ltd**  
 St Anthony's Road  
 Leeds LS11 8DT  
 England  
 Tel: +44 (113) 387 7800  
 Fax: +44 (113) 387 7890

Printed in USA.  
 ©2009 DePuy Orthopaedics, Inc. All rights reserved.

2.5M0409  
 0612-15-506 (Rev. 3)

never stop moving™

